



CHALLENGE SERIES | ZYGOMATIC™

DRILLING SEQUENCE FOR ZYGOMATIC

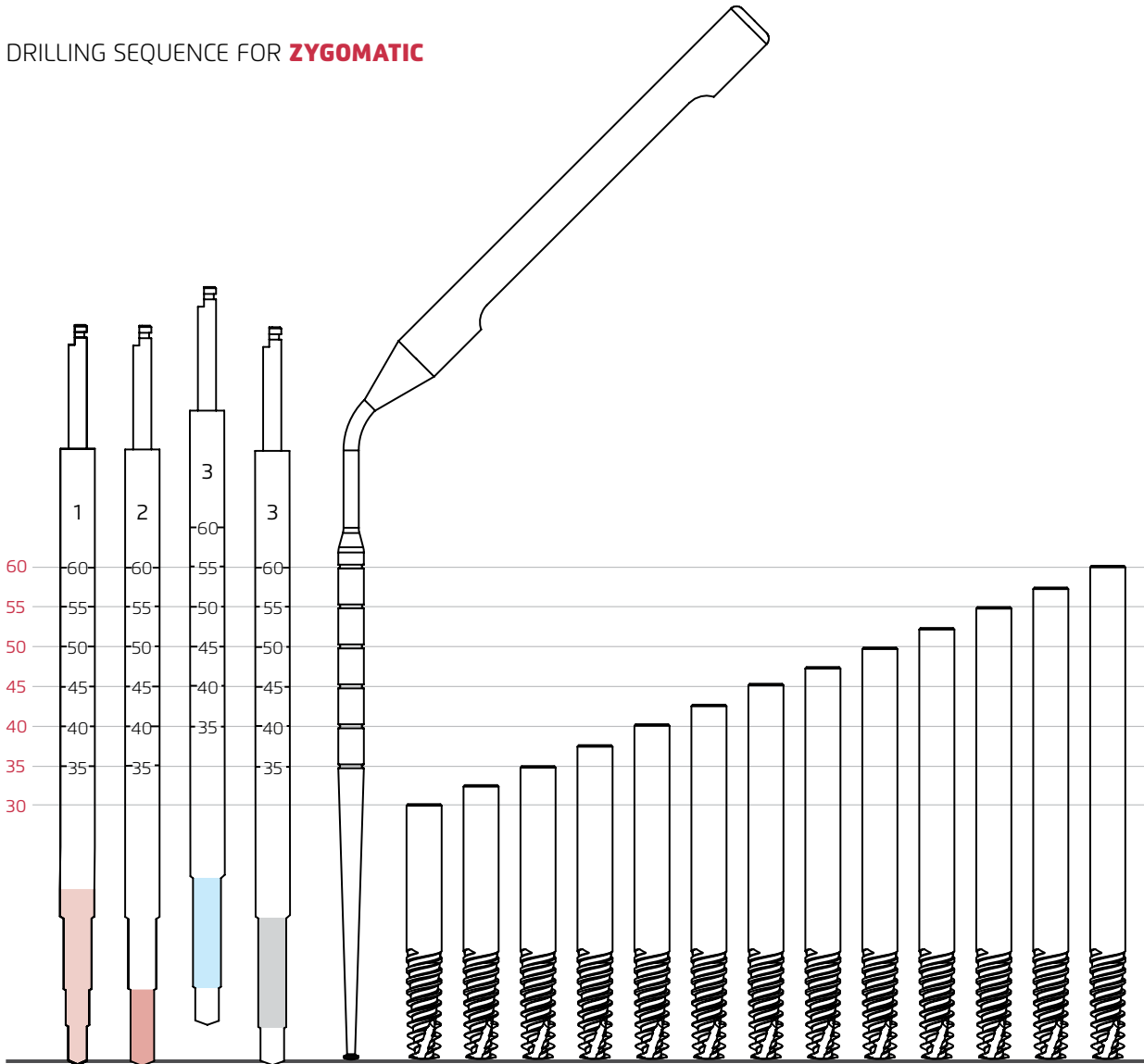

RECOMMENDED STEP DRILL PROTOCOL

IMPLANT DIAMETER	Drill Diameter [mm]	1	2	3
	Drill Speed [RPM]			
	Bone Type			
Ø4.2	D4	↓		
	D3	↓	→ ↓	
	D2	↓	→ ↓	→ ↓ ^{-5mm}
	D1	↓	→ ↓	→ ↓

 Drill osteotomy to implant

 -5mm Drill 5mm Less

Prior to the use of Zygomatic implants in the Zygoma region, additional training is recommended.